

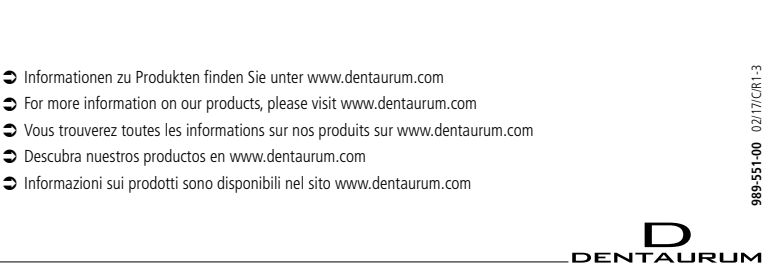
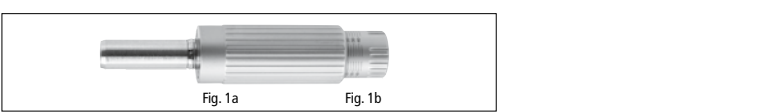
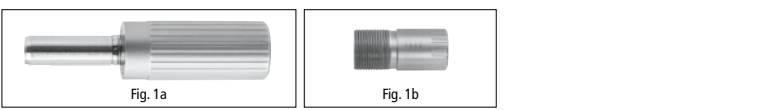
tomas®-handpiece




Turnstr. 31 | 75228 Ispringen | Germany | Tel. +497231/803-0 | Fax +49 7231/803- 295
www.dentaurum.com | info@dentaurum.com

Handgriff mit Kupplung (Fig. 1a) und Einstellschraube (Fig. 1b) | Handle with coupling (Fig. 1a) and adjusting screw (Fig. 1b) | Manche avec coupler (fig. 1a) et vis de réglage (fig. 1b) | Mango con acoplamiento (Fig. 1a) y tornillo de ajuste (Fig. 1b) | Manico con connessione (Fig. 1a) e vite di regolazione (Fig. 1b)

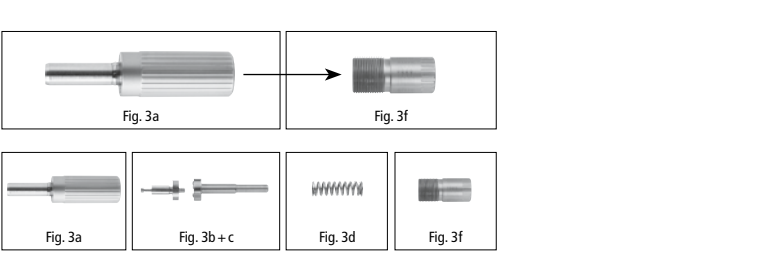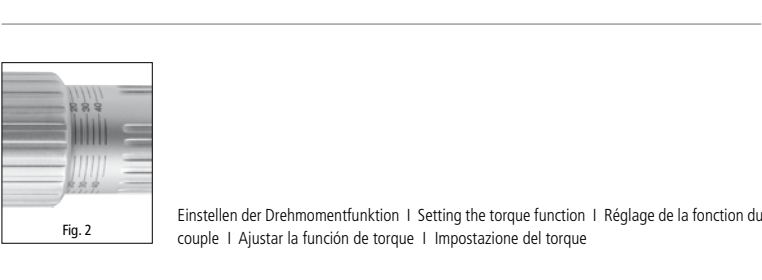Demontage des tomas®-handpiece | Disassembly of the tomas®-handpiece | Démontage du tomas®-handpiece | Desmontaje del tomas®-handpiece | Smontaggio del tomas®-handpiece

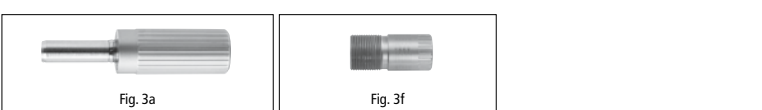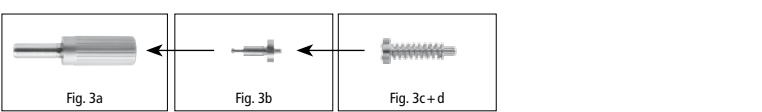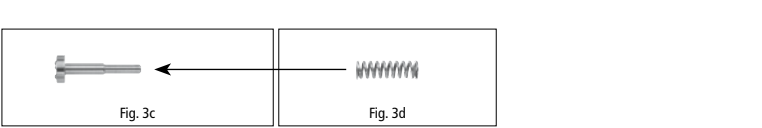Montage des tomas®-handpiece | Assembly of the tomas®-handpiece | Montage du tomas®-handpiece | Montaje del tomas®-handpiece | Montaggio del tomas®-handpiece

4.2 Montaggio

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Thank you for choosing a quality product from Dentaurum. In order to achieve the best results with this product, it is important to carefully study and follow these Instructions for use. Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current Instructions for use supplied with the product and stored in the internet at www.dentaurum.com, even though you may frequently use the same product.

1. Manufacturer

Dentaurum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

2. General product description

Manually turned unit modified to Pauls, with adjustable torque from 0 – 40 Ncm.

The tomas®-handpiece is not sterilized on delivery and must be cleaned, disinfected and sterilized prior to each use, including initial use.

The tomas®-handpiece can be used with standard contra-angled handpieces (blue 1:1). Click the tomas®-handpiece into the contra-angled handpiece.

To insert tomas®-pin: Turn right (clockwise).

To remove tomas®-pin: turn left (anti-clockwise).

3. Using the tomas®-handpiece

The tomas®-handpiece with torque control consists of a handle with coupling (Fig. 1a) and an adjusting screw (Fig. 1b).

3.1 Setting the torque function

The torque set is reached when the respective marking ring of the adjusting screw on the end of the handle is just visible (Fig. 2). When loaded, the coupling releases once the set torque is reached. Do not turn any further, as otherwise there is the risk of exceeding the torque.

 When the tomas®-handpiece is not in use, the adjusting screw should be turned back until the spring is fully released.

4. Maintenance of the tomas®-handpiece

4.1 Disassembly

- Fully unscrew the adjusting screw (Fig. 3f) from the handle with coupling (Fig. 3a).
- Pull the handle (Fig. 3a) and the adjusting screw (Fig. 3f) apart and remove the spring (Fig. 3d) and the two internal shafts (Fig. 3b + c).

4.2 Assembly

- All elements should be cleaned, disinfected and completely dry before assembly. Lubricate the internal shafts (Fig. 3b + c), and the spring (Fig. 3d). This reduces the metal-to-metal friction and prevents frictional corrosion. Only instrument oil (e.g. paraffin-based oil spray) with approval for steam sterilization and tested biocompatibility should be used.
- Place the spring (Fig. 3d) onto the internal shaft (Fig. 3c).
- Insert the internal shafts (Fig. 3b + c) and the spring (Fig. 3d) into the handle (Fig. 3a).
- Wind the adjusting screw (Fig. 3f) into the handle (Fig. 3a) until the resistance of the spring is felt.

 Before sterilization, the tomas®-handpiece should be checked for correct function in both directions.

5. General precautions

5.1 Preparation

Place the tomas®-handpiece in a cleaning/disinfection agent immediately after use. This is for personal safety and prevents contamination from drying. The cleaning and disinfection agent used during this stage is for your personal protection only and cannot replace the disinfection stage to be completed later after cleaning.

The cleaning agent manufacturer’s instructions regarding concentration, immersion time and temperature should be strictly observed.

5.2 Cleaning and disinfection

The tomas®-handpiece must be disassembled for cleaning and disinfection (see 4.1). Cleaning and disinfection can be performed automatically with the aid of a thermodesinfector (adhere to the manufacturer’s instruction) or manually using suitable cleaning and disinfection solutions.

Use of an enzymatic cleaning agent with an almost neutral pH value is recommended. Do not use cleaning and disinfection agents that contain halogens (chlorine, iodine, bromine, fluorine), oxidation agents (H₂O₂), organic solvents, heavy metal salts, aldehydes or aromatic/halogenated hydrocarbons.

 The manufacturer’s instructions should be observed when using an ultrasonic cleaner for disinfection and cleaning.

The following stages are recommended when performing the automatic cleaning / disinfection process in a thermodesinfector:

Cleaning

- Rinse for 1 minute with cold water
- Clean for 5 minutes using a suitable cleaning agent at 55 °C ± 2 °C / 131 °F ± 2 °F (e. g. neodisher® MediClean, 0,5 %)
- Neutralise for 2 minutes in cold water
- Rinse for 1 minute with cold water

Disinfection

- 5 minutes thermal disinfection at 93 °C (199.4 °F) using demineralized water

The following stages are recommended when performing the manual cleaning / disinfection process:

Cleaning

- Mechanical cleaning of the entire surface, particularly of the hollow sections, gaps and drill holes under running water (drinking water, 30 °C ± 3 °C / 86 °F ± 3 °F) with the aid of a soft brush until all residue has been removed (20 seconds minimum per component). Never use metal brushes, abrasive cleaners, steel wool or apply excessive force).
- Clean for 5 minutes in an ultrasonic cleaner (minimum frequency: 35 kHz) using a suitable cleaning solution (e. g. 2 % MediClean®, Dr. Weigert, in drinking water).
- Rinse the entire surface, particularly hollow sections, gaps and drill holes under running water (drinking water, 30 °C ± 3 °C / 86 °F ± 3 °F) for a minimum of 10 seconds per component. Moveable parts must be moved when rinsing.
- Rinse with demineralized water for a minimum of 5 seconds per component.

Disinfection

- Immerse the components for 30 minutes in disinfectant solution (e. g. 2 % neodisher® SeptoMED, Dr. Weigert, in drinking water at 20 °C ± 2 °C / 68 °F ± 2 °F) and move the components in the solution to remove air bubbles on the surface.
- Immerse for 1 minute in cold, demineralized water.
- Thoroughly rinse the entire surface, particularly hollow sections, gaps and drill holes under running water (drinking water, 30 ± 3 °C / 86 °F ± 3 °F) to remove the disinfectant solution residue. Moveable parts must be moved when rinsing.
- Rinse with demineralized water for a minimum of 5 seconds per component.

5.3 Sterilization

The ratchet should be sterilized assembled and unloaded, whereby only thoroughly cleaned and disinfected components without any signs of corrosion may be assembled and sterilized. It is recommended to complete the sterilization using a vacuum autoclave. Pack the tomas®-handpiece in suitable sterilization bags and sterilize using 3x fractional prevacuum for 5 minutes at 134 °C (273,2 °F). Drying time is 20 minutes.

5.4 Storage

Store in a dry environment at room temperature.

6. Warranty

 The operator/product user is responsible for the correct cleaning and disinfection of the product. National regulations and restrictions must be adhered to.

Dentaurum delivers only examined, fault-free products to customers. All of our products are designed and manufactured to satisfy the highest demands on quality. Dentaurum excludes all warranty claims and accepts no liability for immediate or consequential damages caused by:

- misuse
- improper use, application or handling
- improper preparation
- improper maintenance and repair
- non-observance of information regarding use

7. Availability

tomas®-handpiece, 1 piece **REF 302-004-80**

8. Quality information

Dentaurum ensures a faultless quality of the products manufactured by us. These recommendations are based upon Dentaurum’s own experiences. The user is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

9. Explanation of symbols used on the label

 Please refer to the label. Additional information can be found at www.dentaurum.com (Explanation of symbols REF 989-313-00).

Date of information: 02/17
Subject to modifications

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Dentaurum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Allemagne